Safety⁺¹

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The screenshot shows the 'Blood Pressure Judgment Function' (血圧判定機能) settings. The 'Blood Pressure Judgment Mode' (血圧判定モード) dropdown menu is open, showing three options: 'アラート判定する' (Alert Judgment), '乗務判断する' (Crew Judgment), and 'アラート判定する' (Alert Judgment). The 'アラート判定する' option is highlighted, and a hand cursor is pointing at it. The '過去平均を超える場合アラートを表示する' (Display alert when exceeding past average) checkbox is checked. The 'しきい値' (Threshold) is set to 10 mmHg for both '最高血圧' (Maximum Blood Pressure) and '最低血圧' (Minimum Blood Pressure). The '設定基準値を超える場合アラートを表示する' (Display alert when exceeding setting reference value) checkbox is also checked. The 'しきい値' (Threshold) is set to 140 mmHg for '最高血圧' (Maximum Blood Pressure) and 90 mmHg for '最低血圧' (Minimum Blood Pressure). The 'メッセージ' (Message) section contains several messages, including '乗務判断-乗務可能' (Crew Judgment - Crew Possible), '乗務判断-条件付き乗務可能' (Crew Judgment - Conditional Crew Possible), '乗務判断-運行管理者判断' (Crew Judgment - Operation Manager Judgment), '血圧測定開始案内' (Blood Pressure Measurement Start Guide), '血圧再測定案内' (Blood Pressure Re-measurement Guide), '血圧が設定値を超えた場合' (When Blood Pressure Exceeds Setting Value), and '血圧が正常な場合' (When Blood Pressure is Normal). The '保存' (Save) button is visible at the bottom left, and the 'パスワード変更' (Change Password) button is at the bottom right.

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The screenshot shows the 'SAFETY PLUS ONE 環境設定' (Environment Settings) screen. The '接続設定' (Connection Settings) section is active, showing 'COMポート' (COM Port) set to 'COM6', '認証方法' (Authentication Method) set to '未設定' (Not Set), and '血压計機種' (Blood Pressure Monitor Model) set to 'HBP-9020'. Under '血压判定機能' (Blood Pressure Judgment Function), the '血压判定モード' (Blood Pressure Judgment Mode) is set to '乗務判断する' (Crew Judgment). A 'メッセージ' (Message) table is displayed below, listing various status messages and their actions. At the bottom, there is a '保存' (Save) button and a 'パスワード変更' (Change Password) button.

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©2019 AnestSystem. [Build : 1.0.0]